

Bowie & Dick Autoclave Test Pack

INTERSTER 



Bowie & Dick Autoclave Test Pack



After an intense development process Interster International has developed a new Bowie & Dick Autoclave Test Pack which meets all requirements of the ISO 11140-4. The Bowie & Dick Autoclave Test Pack is a Type 2 chemical indicator packed in procurator paper with a cardboard outer packaging which imitates the original Bowie & Dick test. The Bowie & Dick Autoclave Test Pack tests if steam penetration in the autoclave is at sufficient level through colour change of the chemical indicator. The change of colour of the chemical indicator will turn from blue to light pink and indicates if all parameters (time, pressure and steam penetration) are sufficient to approve the process.

Material

The Bowie & Dick Autoclave Test Pack consists of a chemical indicator which is packed in between two layers of procurator paper. The chemical indicator and procurator paper are packed in a special cardboard outer packaging which can be opened very easily after usage. The indicator contains a specially developed indicator ink which is affixed through a silkscreen printing process. All used inks are lead-free.

Advantages


The new Bowie & Dick Autoclave Test Pack has the size of 102 x 76 x 30 mm. Due to the fact the Bowie & Dick Autoclave Test Pack has a smaller size than before, the transport packaging has reduced in size too. The new size therefore will save you costs on both transport and storage.

The Bowie & Dick Autoclave Test Pack has been tested under supervision of the BSI Group. The Bowie & Dick Autoclave Test Pack was positively certified with a BSI Kitemark and completely meets the ISO 11140-4 on all possible Bowie & Dick cycles (B1, B2 and B3)




BS EN-ISO 11140-4:2007

As a result of the starting colour of the Bowie & Dick Autoclave Test Pack being blue and the final colour needed being light pink, a clear contrast is visible between a successful and unsuccessful process. This eliminates any doubt between a positive and negative result of the process. On the image below an unused and used indicator sheet are displayed, the same way they are packed within the Bowie & Dick Autoclave Test Pack.

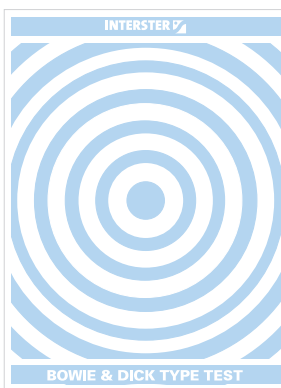

Before use

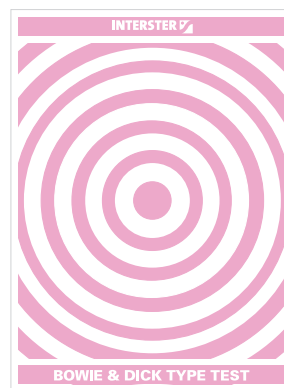

Correct color change*

*This image **cannot** to be used as reference colour for assessing Bowie & Dick Autoclave Test Pack indicators.



Article number	3FSKS610860
Description	Bowie & Dick Test Pack
Unit (box)	25 pieces
Ster. time/temp.	3,5 min. - 134 °C

Interster International B.V.

Samsonweg 2, 1521 RC Wormerveer

T: +31 75 627 22 41 - E: export@interster.nl

www.interster.nl/exporange